

Using Diagnostic Equipment with Eyefinity EHR

Eyefinity EHR works with the diagnostic equipment connected to your computer to import readings into exams or display images that are managed by the equipment's image review software. In some cases, Eyefinity EHR can import information directly from the connected equipment. In other cases, Eyefinity EHR requires that the equipment's software be installed on the computer.

Refer to the *Eyefinity EHR Equipment Integration Guide* for information on installing and using specific pieces of equipment with Eyefinity EHR.

Installing the Equipment Interface


► To install the Equipment Interface

This installer is for users of Eyefinity EHR where OfficeMate is not installed on the computer and where equipment interfaces will be used.

NOTE This installer may not run on Windows 8.1 Professional.


1. Navigate to the folder that contains the EHR Equipment Interface installer.
2. Double-click **EHR_EI_Installer_12.0.3.exe**.

The Welcome window appears.

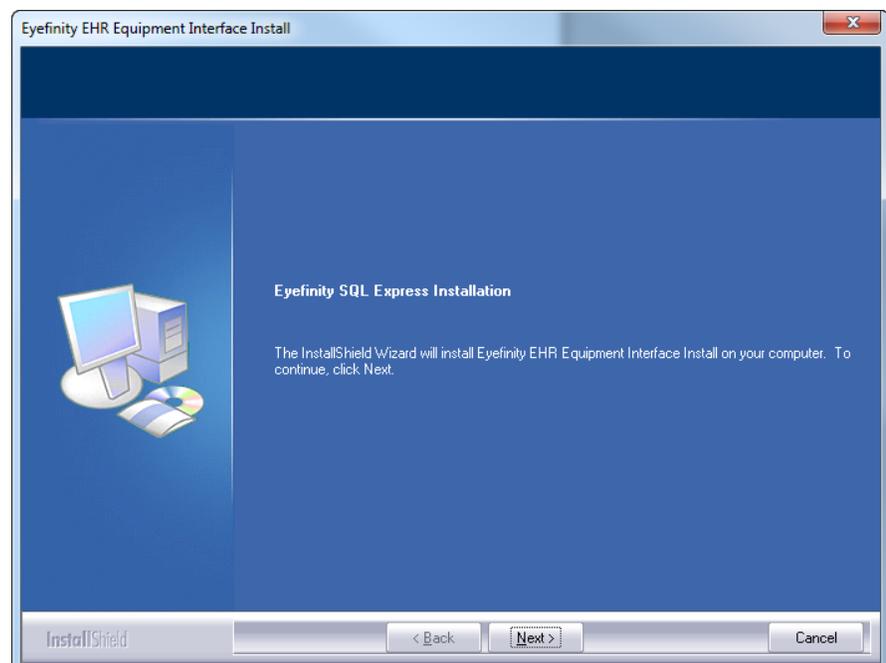

3. Click **Next**.

The Type of Installation window appears.

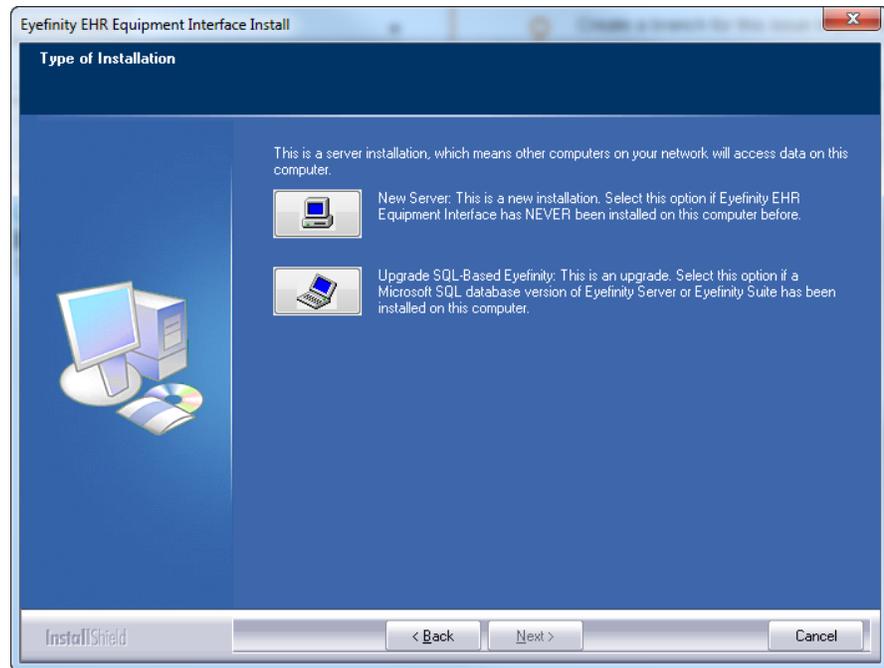


4. Select the type of installation you want to perform.

- **New Server** Select this option if Eyefinity EHR Equipment Interface has never been installed on the computer.
- **Upgrade SQL-Based Eyefinity** Select this option if a Microsoft SQL database version of Eyefinity Server or Eyefinity Suite has been installed on the computer.


5. Click **Next**.

The Choose the disk drive to install SQLExpress on appears.

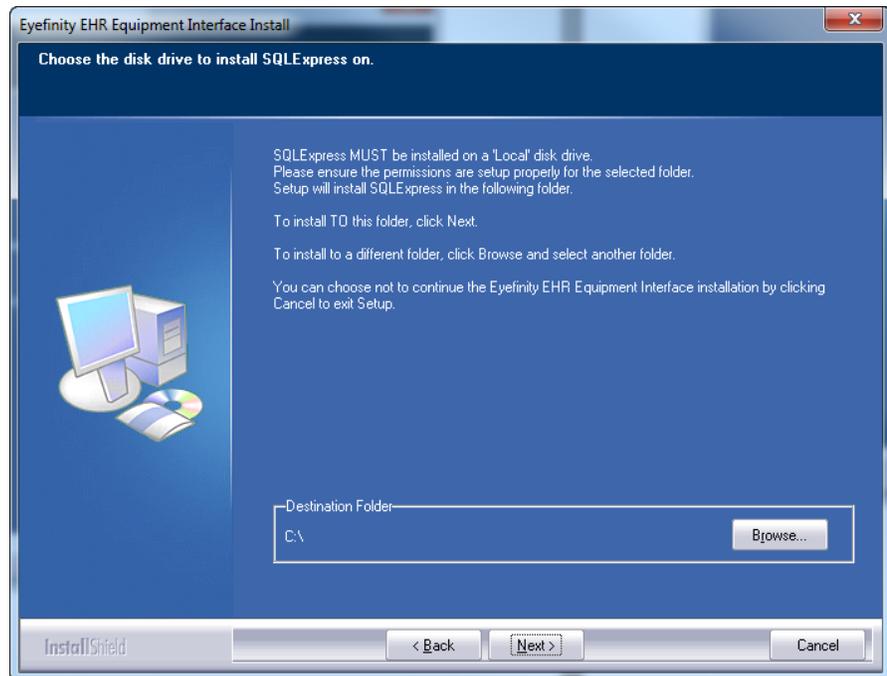

6. Click **Next** to install to the default local disk drive or click **Browse** to navigate to the folder where you want to install the Eyefinity EHR Equipment Interface and then click **Next**.


The Check Eyefinity SQL Setup Information window shows the type of installation that will be performed and the location where the program and data will be installed.

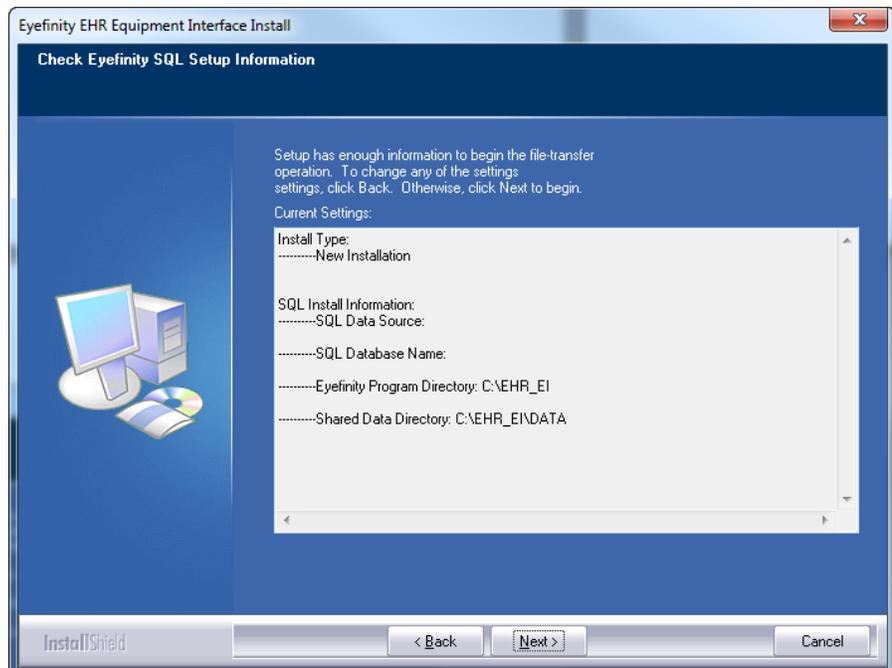


7. Click **Next**.

During the installation various messages appear. This is normal.

The Install Setup Files appears. If an update for EHR Equipment Interface is found and has been selected for installation, the update will be installed.

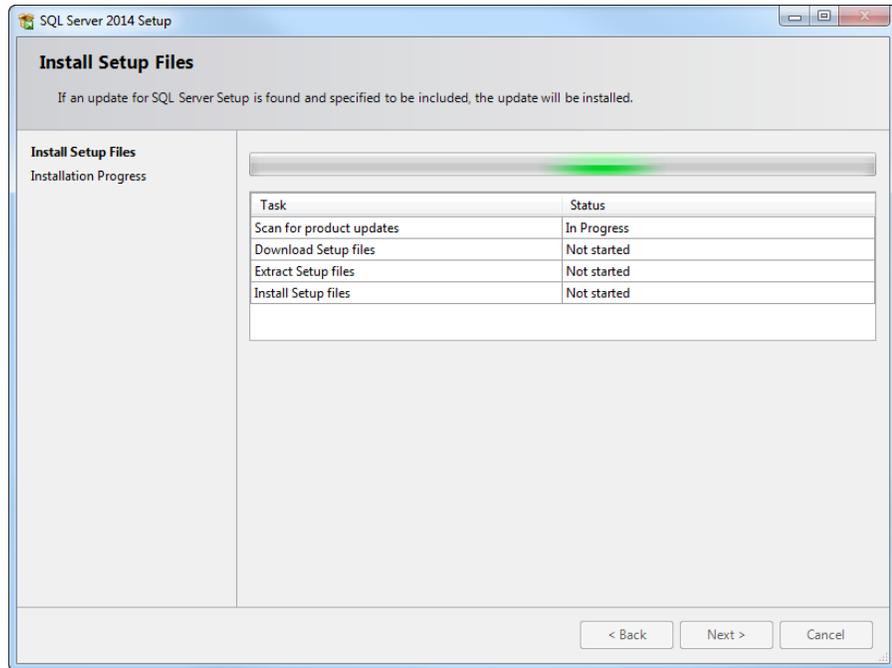


The Installation Progress window appears.

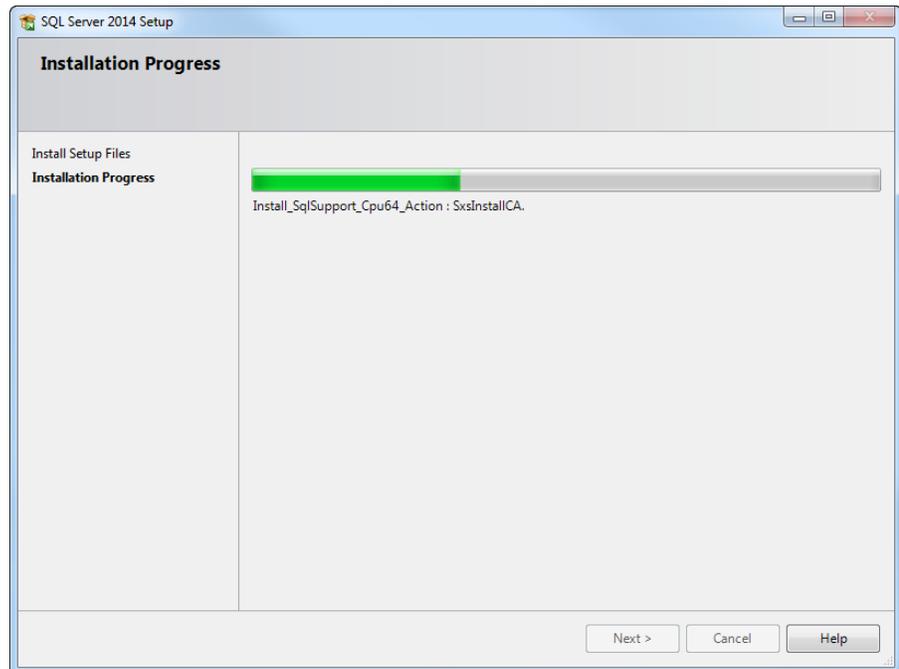

- When the installation is complete, click **Next**.
The Update Eyefinity EHR URL appears.



- Type the web address you enter to access Eyefinity EHR, including the https://. This is the Eyefinity EHR URL assigned to you by the implementation team. Click **OK**.
The Installation Complete window appears.

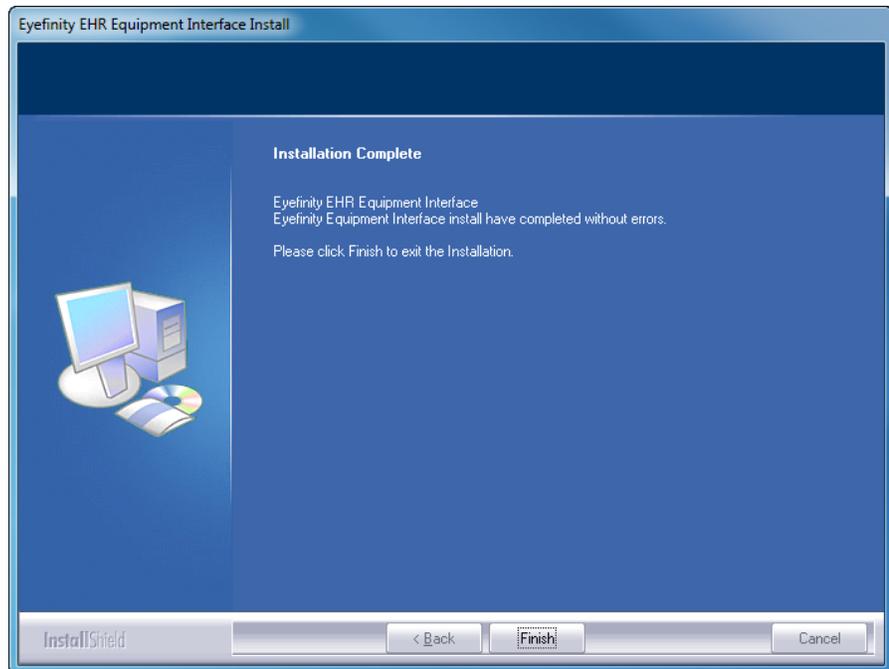

- Click **Finish**.
An icon for the Eyefinity EHR Equipment Interface is available on your desktop.

